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Medtronic

CareLink™ Personal

Welcome to the CareLink™ Personal Therapy Management Software for Diabetes. This Web-based system is designed to help you take information from all of your diabetes management tools – your insulin pump, continuous glucose monitor, blood glucose meter(s), and logbook – and organize it into easy-to-read charts, graphs and tables. These reports can help you and your healthcare provider discover trends and other information that can lead to improved therapy management for greater control.

Medtronic Insulin Pump or Continuous Glucose Monitor → Blood Glucose Meter → Logbook → Medtronic CareLink® Software

Features of the CareLink Personal Software:

- **Personal treatment reports with the information you need**

Dear CareLink™ user, please note that 7 new reports are now available in your CareLink Personal. Just log in and go to the Reports screen.

940M13758-019 20170417
©2017 Medtronic MiniMed, Inc. All rights reserved.

Change country/language

Now everything is at your fingertips. Start today.

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The screenshot shows the Medtronic CareLink Personal web interface. At the top right, the 'Reports' menu item is highlighted with a red box. Below it, a date selection dialog is open, showing a calendar for January 2019. The dates from the 1st to the 14th are highlighted in purple, and this area is also enclosed in a red box. The interface includes navigation links like 'Home', 'My Info', 'Preferences', 'Help', and 'Log-Off'.

2. **SENSOR & METER OVERVIEW** 클릭

The screenshot displays the 'Single Reports' section of the CareLink Personal interface. It lists several report categories: 'Dashboard & Episode Summary', 'Adherence', 'Sensor & Meter Overview', 'Logbook', 'Device Settings Snapshot', and 'Daily Detail'. The 'Sensor & Meter Overview' report is highlighted with a red box. Each report entry includes a small icon, a title, a '(more details...)' link, a brief description, and an 'Add to list' or 'Remove' button.

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